**Bruksanvisning**

SE

**Varning!**

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GB

**Warning!**

The cutting edges of the blades are very sharp and must be handled with extreme care. Never use the tool if you are uncertain of how it is to be used. Store and use out of reach for children.

**Intended Use**

The S-CUT cutting tool is designed for use on patient in rescue situations as well as emergency situations in hospitals. It is a fast way to remove the patients clothes or such like, to expose injuries without changing the patients position that could cause further injuries or unnecessary pain.

**Cleaning**

Disassembly the complete tool. **Do not touch the edge of the blade, it is very sharp!** Clean all parts mechanically (i.e by using a brush or similar accepted method) with a soap solution and water. Disinfect all parts with an 70% alcohol solution and let dry. Make it to a practice to replace the plastic support regularly or when contaminated.

**Maintenance**

Make sure on a regularly basis that the tool is kept well cleaned and not damaged. See illustration A for removal of tool parts. See illustration B for assembly of tool parts. See illustration C and D for replacement of worn parts.

**Contact**

Contact your dealer if you have further questions about this product.

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*Signa Ture* President ES Equipment AB